GlideRite® DLT Stylet

DLT Stylet Instructions for Use **EN**

Stylet DLT Mode D'emploi **FR**

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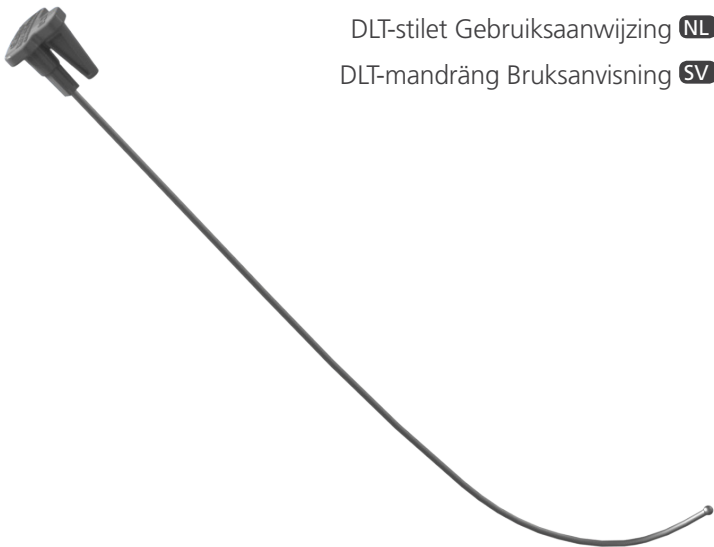
DLT-mandräng Bruksanvisning **SV**

0900-2834-02-60

The GlideRite DLT Stylet was designed to help enable the placement of a double-lumen tube. The rigidity of this reusable stylet helps the user manipulate the tube as desired for intubation. For use in 6.0 mm and larger double-lumen ventilation tubes.

STATEMENT OF INTENDED USE

To provide support for a double-lumen endotracheal tube during intubation.

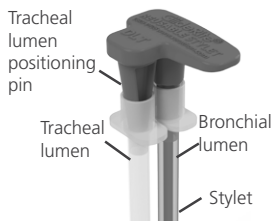
WARNINGS

- The product is nonsterile when shipped and is considered a semi-critical device. Clean and high-level disinfect before first use and between uses.
- Inspect before use—do not use if product is damaged.
- Do not allow the stylet to protrude beyond the distal end of the endotracheal (ET) tube.
- Do not allow the stylet to advance into the glottis or past the vocal cords.
- All cleaning facilities must comply with United States OSHA standard 29 CFR 1910.1030 or an equivalent local standard.

INSTRUCTIONS FOR USE

1. Read all instructions and manuals before using the product.
2. Ensure that the stylet has been high-level disinfected or sterilized.
3. Inspect the stylet for damage. If the stylet is damaged, replace it.
4. Load the GlideRite DLT Stylet into the double-lumen tube through the bronchial lumen. Do not permit the stylet to extend past the distal end of the tube.
5. Rotate the double-lumen tube in order to secure the tracheal lumen on the positioning pin. This changes the natural bend of the tube. The bronchial tip should now aim posteriorly while the tracheal channel is aimed anteriorly.
6. Place the DLT Stylet and tube at the glottic opening with the tracheal lumen facing anteriorly and the bronchial lumen pointing down the trachea. Retract the stylet 5–6 cm (2–3.4 in) with the tracheal lumen facing anteriorly and the bronchial lumen pointing down the trachea as it advances.
7. Completely remove the DLT Stylet and proceed with the intubation using your preferred technique and experience.

For more information, see the *GlideRite DLT Stylet Operations and Maintenance Manual* at verathon.com/product-documentation.

For a full list of symbols used on this and other Verathon® products, please see verathon.com/symbols.

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Copyright © 2017 by Verathon Inc. All rights reserved. GlideRite, GlideScope, Verathon, and the Verathon Torch symbol are trademarks or registered trademarks of Verathon Inc.
Verathon Inc.
Bothell, WA 98011 U.S.A.
Verathon Medical (Europe) B.V.