BLADDERSCAN PRIME PLUS

- EN** CALIBRATION MANUAL
- FR** MANUEL D'ÉTALONNAGE
- DE** KALIBRIERUNGSHANDBUCH
- ES** MANUAL DE CALIBRACIÓN
- DA** KALIBRERINGSMANUAL
- NL** KALIBRATIEHANDLEIDING
- NO** KALIBRERINGSHÅNDBOK
- SV** KALIBRERINGSMANUAL
- FI** KALIBROINTIOPAS
- ZH** 校準手冊
- KO** 칼리브레이션 설명서

BLADDERSCAN®

BLADDERSCAN PRIME PLUS

- EN** CALIBRATION MANUAL
- FR** MANUEL D'ÉTALONNAGE
- DE** KALIBRIERUNGSHANDBUCH
- ES** MANUAL DE CALIBRACIÓN
- DA** KALIBRERINGSMANUAL
- NL** KALIBRATIEHANDLEIDING
- NO** KALIBRERINGSHÅNDBOK
- SV** KALIBRERINGSMANUAL
- FI** KALIBROINTIOPAS
- ZH** 校準手冊
- KO** 칼리브레이션 설명서

Effective: August 8, 2017

Caution: Federal (United States) law restricts this device to sale by or on the order of a physician.

Verathon Inc.

20001 North Creek Parkway
Bothell, WA 98011 U.S.A.
Tel: 800.331.2313 (US and Canada only)
Tel: 425.867.1348
Fax: 425.883.2896
verathon.com

Verathon Medical (Europe) B.V.

Willem Fenengastraat 13
1096 BL Amsterdam
The Netherlands
Tel: +31 (0) 20 210 30 91
Fax: +31 (0) 20 210 30 92

CE0123

Copyright © 2017 Verathon, Inc. All rights reserved.

BladderScan, the BladderScan symbol, Verathon, and the Verathon Torch symbol are trademarks of Verathon Inc. All other brand and product names are trademarks or registered trademarks of their respective owners.

Information in this manual may change at any time without notice. For the most up-to-date information, see the documentation available at verathon.com/product-documentation.

TABLE OF CONTENTS

EN ENGLISH.....	1
FR FRANÇAIS	3
DE DEUTSCHE.....	5
ES ESPAÑOL.....	7
DA DANSK.....	9
NL NEDERLANDS.....	11
NO NORSK.....	13
SV SVENSKA	15
FI SUOMI.....	17
ZH 繁體中文.....	19
KO 한국어	21


CALIBRATING THE SYSTEM

If your facility requires it, you can calibrate your BladderScan Prime Plus system periodically as described in the following procedure. The BladderScan Prime Plus system does not require routine or periodic calibration under normal use. For all other information about how to use and maintain this system, see the Operations and Maintenance Manual available at verathon.com/product-documentation.

PROCEDURE 1. PERFORM THE CALIBRATION

To complete this procedure, you must have a calibration tank designed for the BladderScan Prime system. For more information, contact Verathon® Customer Care or your local representative.

1. Place the calibration tank on a flat, non-reflective surface, and then remove the lid.
2. Using the notches to position the spiral-shaped target correctly, place the target in the tank base.
3. Pour clean, room temperature water into the container, filling to the top of the textured section.
4. Minimize the amount of air bubbles in the water and on the target.

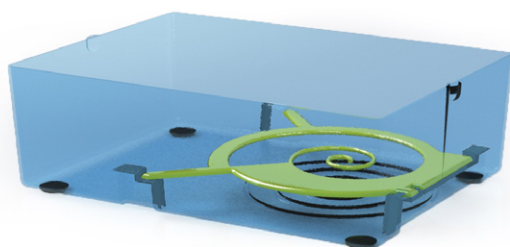
5. Replace the tank lid onto the base.
6. Place the probe into the collar in the tank lid. Align the probe button with the notch exactly as shown, and ensure that the tip of the probe is submerged in the water.

7. On the Home screen, tap **Settings**.

8. Tap **Admin**, and then at the bottom of the screen, tap **Calibrate**.

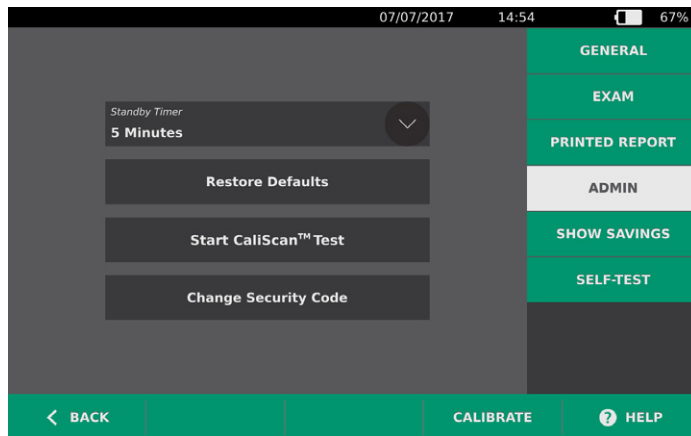9. Review the instructions that appear on the screen, and then tap the **Calibrate** button above them. A waiting indicator appears on the screen while the calibration is in progress.
Note: Once the calibration process has started, do not press the Standby button or disconnect the probe from the console.
 - If calibration is successful, a message on the console indicates that the probe has passed, and the Last Calibration and Calibration Expiration dates are updated.
 - If calibration fails, a message on the console indicates calibration failure.
10. If a calibration failure occurs:
 - Ensure that the calibration tank has sufficient water.
 - Minimize the amount of air bubbles in the water and on the target.
 - Verify that the probe is seated properly in the tank lid.
11. Tap **Calibrate** to begin recalibration. If repeat calibration failures occur, contact Verathon® Customer Care.
12. When you are done, tap **Back**.
13. Remove the probe from the tank lid, and then dry it with a clean, soft cloth.

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