GlideRite

verathon

Rigid Stylet Instructions for Use **EN**

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Statement of Intended Use

To provide support for an endotracheal tube during intubation.

WARNINGS

- The product is nonsterile when shipped and is considered a semi-critical device. Clean and high-level disinfect before use and between uses.
- Inspect before use—do not use if product is damaged.
- Do not allow the stylet to protrude beyond the distal end of the endotracheal (ET) tube.
- Do not allow the stylet to advance into the glottis or past the vocal cords.
- All cleaning facilities must comply with United States OSHA standard 29 CFR 1910.1030 or an equivalent local standard. For more information, visit www.osha.gov.

Instructions for Use

1. Read all instructions and manuals before using the product.
2. Ensure that the stylet has been high-level disinfected or sterilized.
3. Inspect the stylet for damage. If the stylet is damaged, replace it.
4. Load an ET tube (6.0 mm or larger internal diameter) onto the stylet so that the distal end of the stylet does not pass the distal end of the tube.
5. Position the distal end of the ET tube visually above the vocal cords. Do not advance the stylet past the vocal cords.
6. When the ET tube is engaged in the glottic opening, retract the stylet 5 cm (2 in).
7. Position the ET tube per standard practice.
8. Remove the stylet from the ET tube.
9. Clean and disinfect the stylet as described in the *GlideScope and GlideRite Products Reprocessing Manual*.

For More Information

See the *GlideRite® Rigid Stylet Operations and Maintenance Manual* at verathon.com/service-and-support.

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© 2022 Verathon Inc.
GlideRite and Verathon are trademarks of Verathon Inc.
MDSS CH GmbH
MDSS-UK RP Ltd.
Verathon Medical (Europe) B.V.
Verathon Medical (Australia) Pty Ltd