



# Hyperangulated Blade Technique

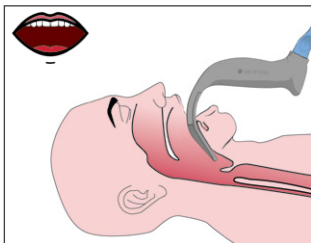


- 1** Insert the blade midline, advancing the tip until it is past the posterior aspect of the tongue.




- 2** Identify the epiglottis and advance the blade into the vallecula for a view of the glottis.

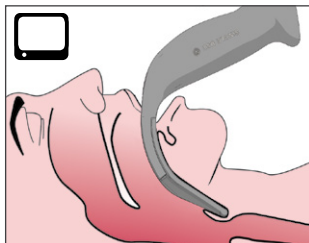


- 3** Using a GlideRite stylet (optional), introduce the ET tube along the right side the blade.

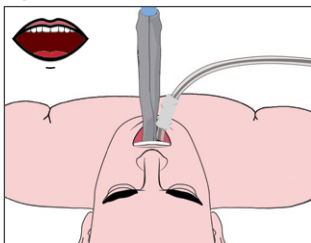


- 4** Complete the intubation.

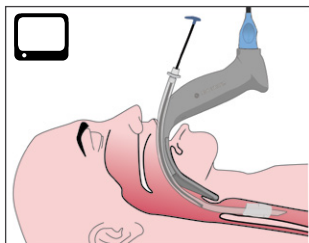


- 5** To remove the GlideRite stylet, pull it out towards the patient's feet.

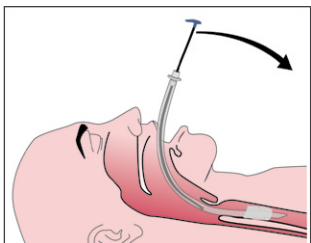

 — Look in the mouth during step.

 — Look at the monitor during step.


 *Important: This document is for quick reference only. For complete instructions, see your Operations & Maintenance Manual at [verathon.com/service-and-support](http://verathon.com/service-and-support).*

 Verathon Medical (Canada) ULC  
2227 Douglas Road  
Burnaby, BC V5C 5A9, Canada  
Tel: +1 604 439 3009  
Fax: +1 604 439 3039

 Verathon Medical (Europe) B.V.  
Willem Fenengastraat 13  
1096 BL Amsterdam, The Netherlands  
Tel: +31 (0) 20 210 30 91  
Fax: +31 (0) 20 210 30 92

Verathon Inc.  
20001 North Creek Parkway  
Bothell, WA 98011, USA  
Tel: +1 800 331 2313 (USA & Canada only)  
Tel: +1 425 867 1348 Fax: +1 425 883 2896

 Anandic Medical Systems AG  
Stadtweg 24  
8245 Feuerthalen  
Switzerland

 MDSS CH GmbH  
Laurenzenvorstadt 61  
5000 Aarau  
Switzerland

Verathon Medical (Australia) Pty Limited  
Unit 9, 39 Herbert Street  
St Leonards NSW 2065  
Australia  
Tel: 1800 613 603 / Fax: 1800 657 970 (Within Australia)  
Tel: +61 2 9431 2000 / Fax: +61 2 9475 1201 (International)

# verathon

GlideRite, Verathon, and their respective symbols are trademarks of Verathon Inc.  
All other brand and product names are trademarks or registered trademarks of their respective owners.  
© 2023 Verathon Inc.



0900-4511 REV-02